ENGLISH - USER MANUAL

Part description

Accessories

Features

- Bone conduction headphones
- Very sensitive microphone
- Dynamic filter to create a perceptive contrast
- Secondary microphone to be held in your hand
- A double-ended jack plug

The voice is simultaneously picked up by the microphone, processed by the dynamic filter in order to induce a perceptive contrast, and transmitted by bone conduction on each side of the temporal bone.


How to wear it

Place the wrap-around headband behind your neck with the transducers resting on the bone in front of your ears, not on your ears! The microphone should be placed in front of your mouth with a minimum distance of 3 cm (1.2 inches). Don't put the microphone too close to your mouth, it could create some feedback!

How to connect to a computer/phone

Using the double-ended jack plug, connect your Forbrain to your computer or phone. This connection to an external source of sound is done in order to practice exercises or complete online sessions. Please note that the source will be amplified, but not filtered.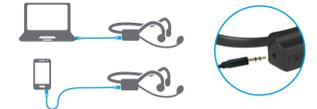

How to connect to secondary mic

Connect Forbrain to the secondary microphone via the jack in front of the black box. Use the additional microphone to practice with the support of a parent or professional. The additional mic should be held in hand at a distance of 5cm from the mouth. The voice of the instructor will be processed as well.

How to charge

Connect Forbrain to a standard USB port on your computer or a wall charger via the USB charging cable. Forbrain will fully charge in 3 hours. The LED indicator will show solid red when charging and the red light will turn off when charging is completed.

LED Indication

Color	Status
Solid Blue	Normal working mode
Blue light turns off and power off	Battery too low
Solid Red	Headphone is being charged
Red light turns off	Charging completed

Troubleshooting

Issue	Resolution
Headset does not turn on	Check if the battery is fully charged. If the battery is not charged, do so.
I cannot hear my voice or the sound is too loud	Check the position of transducers, microphone, & volume level. Position the transducers correctly and switch on and on the device to reset to default volume. Then adjust the volume until you are comfortable.
I noticed changes in intensity of the feedback of my voice	This is not an issue. This happens due to the dynamic filter that processes your voice in real time.

Specifications

Speaker Type	Dual suspension bone conduction
Sensitivity	100 ± 3dB
Microphone type	Back Electret Condenser
Sensitivity	-60dB ± 3dB
Playback time	Up to 6 hours
Charging time	Around 3 hours (with USB cable)
Weight	60gr
Color	Black with blue transducer pads
Carton size	190x165x63mm

Storage and maintenance

- Store the headset in cool and dry place. The working temperature should be 0-45°C (32-113°F). Working in environments that are either too hot, too cold, or too humid, will reduce the battery life.
- After storing for a long period, please charge the headset before using again.
- The headset is not water resistant. Do not put the headset into water.
- Clean the headset with a soft, dry cloth.
- Do not bring the headset into contact with any sharp objects.

Note:

Design and specifications are subject to change without notice. For more information on the product, please visit www.forbrain.com

FCC Warning

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) this device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

Where is the serial code?

Precautions, limitations, and disclaimer

You do not need to change the volume of Forbrain with a few rare exceptions (very poor or hypersensitive hearing). Do not turn the volume of Forbrain up too high. You should feel comfortable. Children must use Forbrain under constant adult supervision and instruction. Do not use Forbrain more than 2 hours a day. If you feel pain or discomfort, stop using Forbrain immediately. Resume using Forbrain once you feel well again. Forbrain is an electronic device, use it carefully. Forbrain is a pedagogical device. It is not approved by the Federal Drug Administration or other health organization. It cannot be considered as a medical device or an accessory to a medical device.

How to contact us?

SOUND FOR LIFE Ltd
Unit 507, 5/F, Chinachem Golden Plaza, 77 Mody Road,
Tsim Sha Tsui East, Kowloon,
Hong Kong
E-mail: info@forbrain.com

Note:

If you need to return a product under warranty, please send a detailed explanation of the problem to: info@forbrain.com

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