TalksUp®-BT

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Content of a TalksUp®-BT package

- 1x TalksUp®-BT player
- 1x USB flash drive containing the Program Manager software
- 1x USB charger and 1x USB cable
- 1x 3.5mm jack cable
- 1x Large rigid case designed to hold the equipment
- 1x Small rigid case to protect the TalksUp®-BT from shocks and a belt
- 1x Silicone case to protect the TalksUp®-BT when in use
- 1x headband

Read carefully before using the TalksUp®-BT

- TalksUp®-BT was designed to work with the Program Manager software supplied by TDSA. Program Manager does not work on Mac OS.
- **Program Manager only works with Windows computers**
- Before you use it for the first time, copy the content of the USB flash drive onto your computer desktop and watch the explanatory video.
- TalksUp®-BT is delivered without any programs installed. You have to transfer programs using the Program Manager software.
- Do not use any object such as a stylus on the touch screen.
- If the screen has not been used for 30 seconds, the backlighting goes into energy-saving mode to optimize battery life.
- The touch screen automatically locks to block any commands during a listening session.
- To reactivate the touch screen, place a finger on the screen and swipe.
- When playback stops and the screen is not touched for 3 minutes, the device automatically switches off to save battery life.
- The TalksUp®-BT functions with a battery that has a maximum life of 8 hours (varying according to use and age of the device).
- To recharge the TalksUp®-BT battery, **use the USB charger supplied by TDSA**. A non approved charger may damage the device.
- It is advised to charge the device when the battery indicator displays 2 green squares or less. When recharging is required, a “Please charge device” message appears on the TalksUp®-BT home screen.
- The device must be recharged by connecting the USB charger to a power outlet.
- **The TalksUp®-BT should be powered on for charging.**
- The TalksUp®-BT will automatically turn off once fully charged.

Description of the TalksUp®-BT

Mini USB port for recharging the battery by connecting the USB cable with a charger plugged into a power outlet

ON/OFF button

Mini jack to connect a microphone

Mini jack to connect wired headphones or wireless headphones transmitter

External card reader to insert a MicroSD memory card (not included) containing specific audio tracks for TalksUp®-BT

Getting started

- Press then release the start button to switch on the TalksUp®-BT. The start screen should appear.

- Connect the wired headphones to the mini jack.
- OR
- Pair your BT-Headphone to the TalksUp®-BT. (See page 8)
- After a few seconds, the start menu displays two options: "Programs" or "Settings".

TalksUp®
powered by Tomatis®

Start screen

Welcome to your **TalksUp®**

Programs

Settings


Start menu

The "Settings" menu

The screen opposite displays the following information:

- The current date.
- The TalksUp®-BT serial number.
- The expiration date for the validity of the current license.
- The name of the device owner.
- The version of the micro-software installed (FW).
- The horizontal "Display brightness" cursor lets you adjust screen brightness with your finger.


Please note: Increasing screen brightness affects battery life.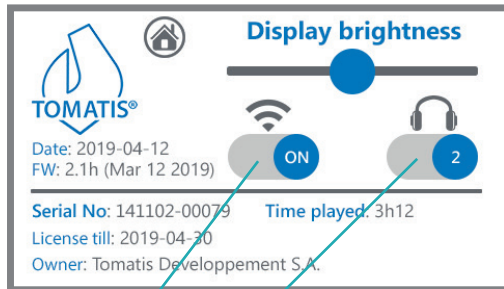

Number of headphones wanted in BT mode (see page 8)


Activating BT (see page 8)

The "Programs" menu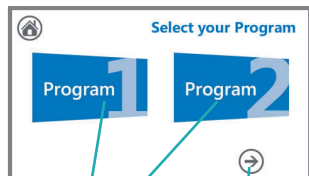

Program selection

Next programs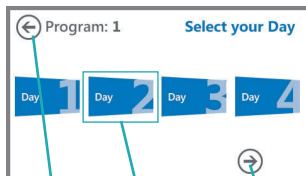


Previous page

Next days


The green rectangle highlights the day to be played

Please note: If the programs have been transferred using the Forward mode, the following days will be shown in gray.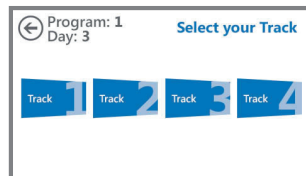

Please note: If the programs have been transferred using the Forward or Replay modes, the following tracks will be shown in gray.

Playback screen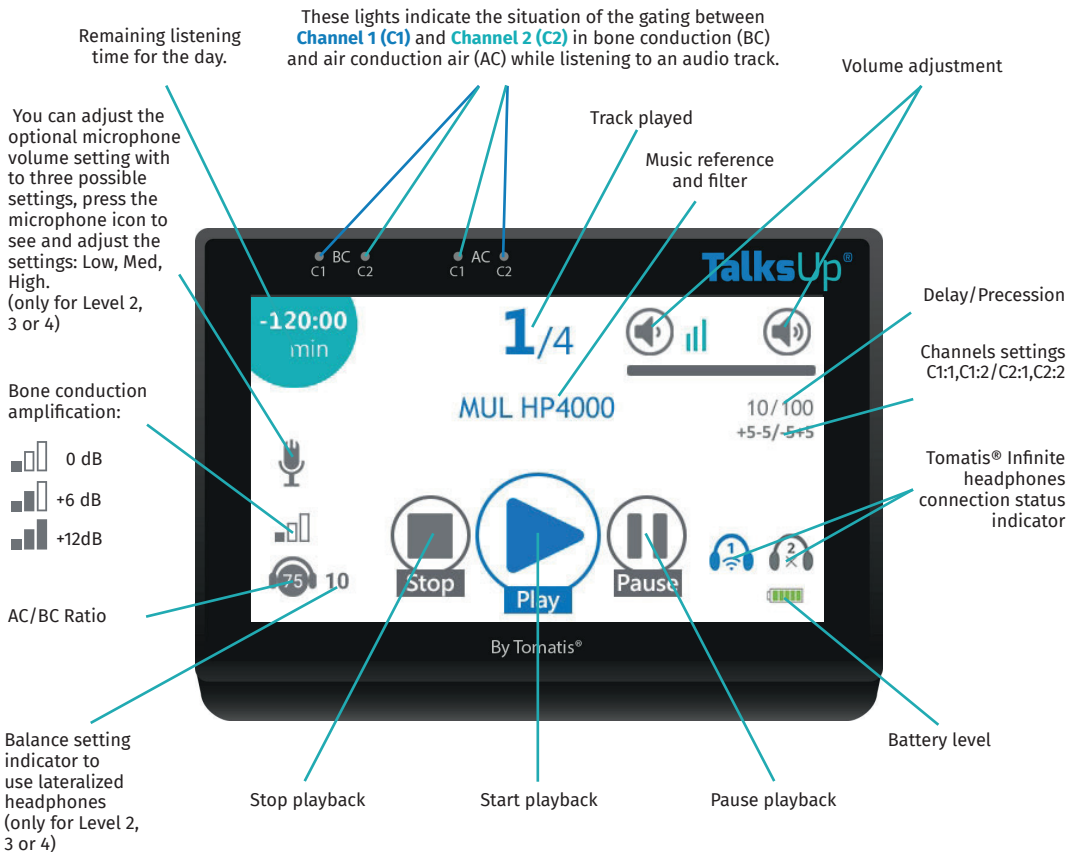

Playback screen

- If you click on the “Pause” icon during a listening session, the music stops until you click on “Pause” or “Play” a second time.
- If you click on the “Stop” icon while listening to a track, the message “Confirm Stop” is displayed.
- Click on “Stop” a second time to stop the session playback.

Please note: this will definitively stop the playback and prevent you from resuming it.

- When the listening session is done, the message “Session done” is displayed on the screen.
- To switch off the screen while listening to a track, hold down the on/off button for 2 seconds. This extends battery life. You must again press on this button for 2 seconds to switch the screen back on.
- To switch off the device, go back to the Start menu and hold down the on/off button for 3 seconds.

Charging the battery


When the device is charging on Alternating Current (A/C), the battery charge indicator can change to display the following situations:

: the TalksUp®-BT battery is recharging

: the TalksUp®-BT is plugged into an A/C outlet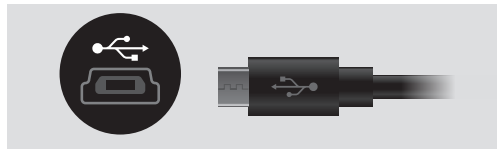

To avoid damaging the battery, be careful not to let the device discharge completely, especially when it is unused for a few months.

Audio tracks

MU

MUA	MUHA	MUP	SMUE
MUB	MUI	MUPM1	SMUK
MUCL	MUJ	MUPM3	SMUO
MUCLA	MUK	MUQ	SMUS
MUD	MUL	MUS	SMUCL
MUE	MUM	MUT	SMUCLA
MUF	MUN	MUV	
MUG	MUO	SMUD	

ACTIVES (FROM LEVEL 2)

GR2D	VGB3	CRS2	GR1HF
GR2E	VGC1	CRS3	GR1LA
GR2F	VGC2	CRS4	GR1LD
VGA1	VGC3	CRS5	GR1LE
VGA2	2GR1H	CRS6	GR1LF
VGA3	2GR1L	GR1HA	GR2A
VGB1	2GR2	GR1HD	
VGB2	CRS1	GR1HE	

GFM+GM

GFM1	GFM10	GM27
GFM2	GM3	GM28
GFM3	GM7	GM32
GFM4	GM11	GM33
GFM5	GM12	GM34
GFM6	GM16	GM37
GFM7	GM19	GM38
GFM8	GM25	
GFM9	GM26	

VL, MD and others

MAR1	VL1
MAR2	VL2
MAR3	VL3
MD1*	VL4
MD2*	VL5
MD3*	VL6
MD4*	
MD5*	

This list does not concern the IL/AS-ENA TalksUp® *For level 4 only

Tomatis® Infinite headphones connection and pairing

1. On your TalksUp®-BT go to «Settings» and switch the button from OFF to ON .

2. Enable pairing detection on your Tomatis® Infinite headphones. Make sure that the Bluetooth mode is turned on.

3. Click on the  icon and then click on Analyze, wait 10sec.

4. Click on your headphones address on the list and click on the grey round , the headphones icon should be blue after a few seconds.

5. Start your program

If you want to pair 2 headphones, switch the number button from 1 to 2  on the setting menu, click on the

1st headphones address on the list and click on  (first column).

Repeat the operation for the second headphones, clicking  on the second column. After a few seconds the two icons should be blue  .

If the headphones disconnect during a session, click on the Pause button, turn back on the headphones and click on this icon  to re-pair the headphones . Press play again to continue the listening.

Once your session is over, don't forget to turn off the TalksUp-BT and the Tomatis® Infinite headphones.

Please note: If you own a regular TalksUp® you will need a bluetooth receiver (aptX) to use it with the Tomatis® Infinite headphones.

Legal warranty

- The TalksUp®-BT pack is commercialized in the framework of the use of the Tomatis® Method with a 1-year legal warranty covering manufacturing faults or any dysfunction in the context of normal use.
- The warranty does not cover damage caused by shocks, contact with a liquid substance, wrongful use or use of TalksUp® with inappropriate equipment.
- If the sticker on the back of the TalksUp®-BT indicating

the serial number has been removed or has deteriorated, the warranty will be cancelled.

- Should you encounter difficulties with the device or wish to request a repair, please send an email to info@tomatis.com describing the problem encountered and specifying the serial number of the TalksUp®-BT player, as indicated on the back of the device or in the Settings menu.
- TalksUp®-BT is a pedagogical product and may in no way be considered as a medical device.

Option: Microphone for Active work

To use the device in active mode, Practitioners (Level 2) and Consultants can purchase a microphone.

Omnidirectionnal microphone

Please note: In order to use the microphone, the headphones need to be wired to the TalksUp®-BT. They won't work in Bluetooth mode.

TalksUp®-BT Specifications

Frequency Response 80Hz-20kHz @ -10db (exclusive int. Filter setting)	+1.5/-1.5 dB
S/N Ratio @-10db	>-90 dB
Headphones Connector	3.5 mm TRS
Headphones Output Impedance	16-32 Ohms
Maximum Output RMS @ 32 Ohms	30 mW
Microphone Connector	3.5 mm TRS
Microphone Input Impedance	47k Ohms
Mic Input Gain Range (internal trim):	-26 to -46 dB
Microphone-Type	OM01 - Omnidirectional Condensor Microphone by Tomatis®
Maximum Input/Output Codec/A/D-D/A:	0 dB
Sampling Rate:	48kHz
A/D—D/A Converters:	16-bit
Accumulator Operating Time	6-8 hours
Power-Supply (charging – max.)	USB 5V DC 1.6A
(operating – avg.)	5V DC 0.25A
Optional Memory	Micro SDHC - 32 GB minimum (class 6 or higher)
USB Data-Connector	Mini-USB 2.0
Weight	270g/~0.6lbs
Dimensions Height/Width/Depth	83 mm/117 mm/21 mm
TalksUp®-BT Program Manager Environment	Microsoft Windows (XP, Vista, 7, 8, 8.1 and 10), 64 or 32bits
Min. Resolution	1024 x 768

BT Specifications

BT Standard: 4.0 Class 2 apt-X | Frequency Band: 2,402 MHz to 2,480 MHz |
RF Sensitivity 0.1% BER at -88dBm | Transmit Power BER/EDR Class2<4dBm,
BLE<10dBm | Modulation 8 DPSK, PI/4 DQPSK, GFSK | Maximum Data Rate
3Mbps (typical 1.6Mbps)

Audio Specifications

DAC resolution: 16 bits | DAC Output Sample Rate: 8 KHz to 90 KHz | DAC SNR: 96dB |
Stereo Separation: -87.7dB

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TOMATIS DEVELOPPEMENT S.A.